# Lumina Max LAD

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## Thank you for choosing Lumina Max LAD Precision-Tech.

All Precision-Tech brand products are guaranteed by Dentalica.

A dentist's work is to take care of his clients, ours is to take care of every dentist.

We select the best options, we test and make them our own. Our experience guarantees their quality, both before and after the purchase.

Please do not hesitate to contact us if you have any further questions on this manual, in case of need: Toll-free number 800.707172 - info@dentalica.com

We recommend to keep this manual in a safe place for future reference.

The manufacturer reserves the right to modify the information and the data provided in this manual when necessary and with no obligation to notify.

### SYMBOLS USED ON INSTRUCTIONS

#### **Caution**

*If the instruction are not being followed properly, operation may result in hazards for the product or the user/patient*

### SYMBOLS USED ON PACKAGING, DEVICE AND PARTS

Consult Instructions for Use

Product Serial Number

Catalogue number identification

Manufacture identification number

Expiry date

Medical Electrical Equipment Directive, DS/EN 60601-1:2006  
Electrical shock protection, Type B

Medical Electrical Equipment Directive, DS/EN 60601-1:2006  
Electrical double insulation, Class II

Medical Devices Directive, MDD Directive 93/42/EEC  
Medical device, Class I

Canadian Standards Association, CSA  
Approved for use in Canada and USA

Waste Electrical and Electronic Equipment Directive, WEEE  
Directive 2002/96/EC. Dispose product to a recycling central

## TABLE OF CONTENT

<b>1. Technical description</b>	.....pag. 13
1.1 Technical specifications	.....pag. 13
1.2 Environmental conditions for intended use	.....pag. 13
<b>2. Function and Use</b>	.....pag. 14
2.1 Connecting	.....pag. 14
2.2 Buttons	.....pag. 14
2.3 Program modes	.....pag. 14
Light Curing – Green Program	.....pag. 14
Light activated disinfection of surface - Orange Program	.....pag. 15
Root canal and periodontal disinfection - Red Program	.....pag. 15
<b>4. Indications</b>	.....pag. 16
4.1 Light indication signals and sounds	.....pag. 16
4.2 Temperature safety	.....pag. 17
4.3 Battery power saving	.....pag. 17
<b>5. Accessories</b>	.....pag. 17
5.1 Cover and tips	.....pag. 18
5.2 QroxB2® – Disinfection Agent	.....pag. 18
<b>6. Maintenance</b>	.....pag. 18
6.1 Barrier Sleeves	.....pag. 18
<b>7. Service and warranty for Italy</b>	.....pag. 18
<b>8. Disposal of waste products</b>	.....pag. 18

# 1. TECHNICAL DESCRIPTION

The Lumina Max LAD intended usage is light activation of photosensitive materials in the light spectrum around 460 nm. Examples are dental composites and other dental light cured materials.

Furthermore, the Lumina Max LAD light can be used for light activated disinfection with photosensitizing agent QroxB2®.

The product is classified as an electronic medical device electric class II according to DS/EN 60601-1:2006. The product is classified as a class 2 LED product according to EN 60825-1:2001 Ed 1.2.

The manufacturer declares that the product described in this manual complies with the above mentioned standards.

## Warnings

1. Do not expose directly into the eyes, or look directly at the emitted light.
2. It is recommended to use eye protection.
3. Medical electronic equipment is sensitive to electromagnetic fields. Portable and mobile high frequency communication devices may interfere with medical devices.

4. In order to maintain proper operation, follow instructions in this manual.
5. Use of the light without a tip may cause overheating of the tissue if the light is used close to the gingiva.
6. Extended use cause tips and cover to be hot and may cause slight burning sensation of mucosa, if touched.
7. Do not reuse covers or tips as this may cause cross contamination.
8. If the light beeps every 2s when placed in docking station, it is a warning that the charging voltage is 6V or higher, in other words a wrong power supply is being used.
9. The Lumina Max LAD photosensitizer QroxB2® can stain clothing.

## 1.1 Technical specifications

The Lumina Max LAD light consists of a hand piece, a docking station and an external power supply. Disposable covers and tips protect against cross contamination.

POWER SUPPLY	
Manufacturer	Dongguan Shilong Fuhua Electronic Co. Ltd.
Model	UEO8WCP-050100SPA
Input power	100–240VAC, 50/60Hz
Output power	5.0VDC, 1.0A
Or equivalent medical approved power supply supplied by the manufacturer	

BATTERY	
Battery type	LiFePO <sub>4</sub>
Nominal voltage	3.2V
Nominal capacity	1.450mA
Re-charging time	2,5 hours
User capacity	approx. 760 seconds

LED DIODE	
Output intensity	5.500-7.500 mW/cm <sup>2</sup>
Wavelength	450-470nm (85%), peak 460nm

## 1.2 Environmental conditions for intended use

Temperature range	+10°C to +40°C
Humidity range	30% to 75%

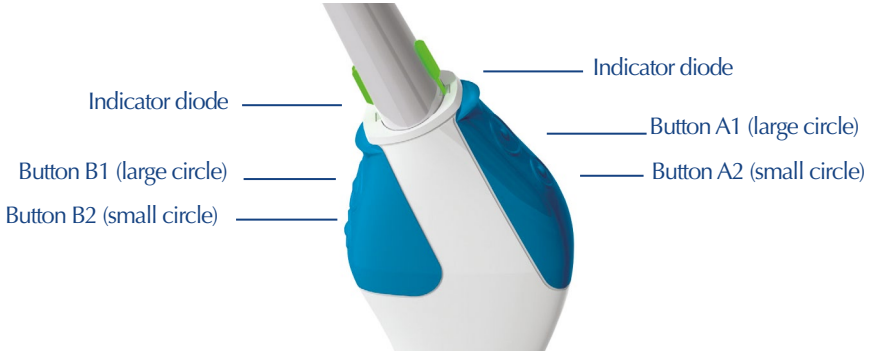
Atmospheric pressure range	70kPa to 106kPa (0.7 bar to 1.06 bar)
IP class	2X

## 2. FUNCTION AND USE

### 2.1 Connecting

Connect the power supply under the bottom of the docking station and to a main electrical outlet. Place the light in the dock and the battery will begin charging. This is indicated by a green light flashing.

### 2.2 Buttons

Button function: A1 = B1

Button function: A2 = B2

### 2.3 Program modes

The Lumina Max LAD light can be used for multiply intended uses. Each type of treatment has its own program. The set program is shown by the color of the indicator diode when the light is in program mode. Programs are Green, Orange or Red program.

To enter program mode, press the two buttons on the same time and hold for 3s until a beep sounds. Scroll by pressing either button and stop at the desired program, stop pressing the buttons and wait for the beep. The light will stay in the chosen program until manually changed.

#### Light Curing – Green Program

The green program mode is used for curing composites and other dental materials.

1. Cure with the light (tip) as close to the composite as possible, as light intensity is reduced by distance.
2. Build the restoration in multiply layers of 2-3mm composite and cure each layer for 1s or 3s.
3. If each layer is wider that the tip – move the tip and cure again.
4. Always cure the final layer for 3s.

SMALL BUTTON	LARGE BUTTON
1s 	3s 

#### Caution

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
Never light directly on gingiva without a tip, as the heat may damage the tissue. If the composite is darker than A3, use 3s for each layer as penetration of the light is reduced in darker materials.

## Light activated disinfection of surface - Orange Program

The orange program is used for photo activated disinfection with the light tips touching the gingiva. The 8 mm blunt tip must be used for this purpose, otherwise the tissue may be overheated. In this program the light vibrates when activated to ensure micro movements in the fluid/gel.

*Use - Gingivitis or perimucositis*

1. Apply QroxB2® Perio in the pocket and on edge of the pocket.
2. Activate Light for 5s per treatment spot.

SMALL BUTTON	LARGE BUTTON
5s  	10s  

### Caution

Never treat more than 5s at same spot to avoid a burning sensation. If using the 10s setting, move the tip from side to side.

*Use - Mucositis or other surface inflammations*

1. Apply the QroxB2® Perio over the entire inflamed area using for example a cotton pellet.
2. Activate Light for 5s per treatment spot.

### Caution

Never treat more than 5s at same spot to avoid a burning sensation. If using the 10s setting, move the tip from side to side.

## Root canal and periodontal disinfection - Red Program

This mode must only be used with either a perio or endo tip attached.

Otherwise, the heat emission is too high and may provoke tissue damage. In this program the light vibrates when activated to ensure micro movements in the fluid/gel.

*Use - Root canal disinfection*

Preparation: Clean and shape the root canal using copious amounts of sodium hypochlorite, citric acid or similar. Dry.

1. Fill the canal with disinfection agent QroxB2® Endo.
2. Insert the endo light tip until resistance is met and activate light 2x30s.
3. Rinse and dry.

SMALL BUTTON	LARGE BUTTON
15s  	30s  

*Use - Periodontal disinfection*

Preparation: Perform conventional scaling and root planning.

1. Fill the pocket with disinfection agent QroxB2® Perio.
2. Treat pocket from inside (perio tip 15mm or 23mm) minimum 15s per surface of tooth, depending of depth. See table.

Situation	Dosage
PD > 6 mm	15s per site
PD > 8 mm	30s per site
PD > 10 mm	45s per site
Apical periodontitis	Extra 10s on gingiva at level of apex

3. Rinse pocket.

## 4. INDICATIONS

### 4.1 Light indication signals and sounds

<b>READY</b>		Ready for use.
<b>ACTIVATED LIGHT</b>		Sound to indicate start and stop. <b>DANGER!</b> Do not stare into light. It may cause eye damage.
<b>CHARGING BATTERY</b>		May be used.
<b>LOW BATTERY</b>		If not used, place in dock.
<b>DIODE HEAT WARNING</b>		15s from cut-off.
<b>DIODE OVERHEATED</b>		Needs to cool before it can be activated. <b>DANGER!</b> Tip is hot and may cause light burning to mucosa.
<b>SLEEP MODE</b>		Press any button.
<b>DEEP SLEEP MODE</b>		Place in dock.
<b>ERROR</b>		Reset by pressing two buttons on same side for 3s. <b>DANGER!</b> No effect of treatment.

## 4.2 Temperature safety

The Lumina Max LAD light has a safety cut off to preserve the lifetime of the diode should overheating occur. When the light gets close to the cut off temperature, an intermittent beep warning occur. The light can be used for approximately another 15s before it cuts off. Overheating is indicated by 3 short beeps followed by fast red flashing.

## 4.3 Battery power saving

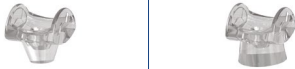
### Sleep mode

If the light has not been activated for 5 minutes or placed in the dock, it will go into Sleep mode to conserve energy. Press any button to reactivate it.

### Deep Sleep mode

If the light is not placed in the charging dock or activated for 6 hours the light will go into Deep Sleep mode. It must be placed in the dock for one second to be reactivated.

## 5. ACCESSORIES

<b>COVER</b> PRO0LMCO			Covers protects the light from contamination and the treatment tips connect to the cover. Covers are single-use and disposable.
<b>BLUNT TIPS</b> <b>Ø 4 mm</b> PRO0LMP4	<b>BLUNT TIPS</b> <b>Ø 8 mm</b> PRO0LMP8		The blunt tips is used for surface disinfection treatment and when curing composites close to the gingival. This is to avoid too much heating of the gingiva, which may happen when the light is used without tips.
<b>PERIO TIP</b> <b>L 23 mm</b> PRO0LM23	<b>PERIO TIP</b> <b>L 15 mm</b> PRO0LM15		The perio tips is used for periodontal treatment. The perio tips have a diameter of 1 mm. The distal 5 mm have a surface permitting the light to go sideways.
<b>ENDO TIP</b> PRO0LMEM			The endo tip is used for endodontic treatment. The endo tip has a diameter of 0.5 mm.

## 5.1 Cover and tips

There are five tips available for the Lumina Max LAD. Use the tip that suits the desired treatment best. Tips do not fit the light without the Cover. Tips snap onto the Cover by pressing the two parts together. Tips can be positioned in an +/-15 degrees angle in the longitudinal direction of the cover. To detach the tip, rotate the tip away from the cover and it will pop out.

## 5.2 QroxB2® – Disinfection Agent

To use the Lumina Max LAD light for disinfection purposes it requires the QroxB2® Perio or QroxB2® Endo disinfection agent.

QroxB2® Perio is a gel formulation for use in pockets and on the tissue surface. ( PR00LMBP )

QroxB2® Endo is a liquid formulation for use in the root canal. ( PR00LMBE )

The concentration of active substances are the same, the gel is just easier to use in pockets and on surfaces. Both disinfection agents comes as a powder in a syringe.

To prepare the disinfection agent draw up of the amount of water as indicated on pro-duct label of QroxB2® and shake until the powder is dissolved. For the gel formulation, wait 1min for the gel to have the proper viscosity.

## 6. MAINTENANCE

The hand piece and docking station may be wiped with a slightly damp cloth.

### Caution

- Use only 70% alcohol when cleaning the light and docking station.
- Make sure NOT to leave fluid in the charger socket, remove excess fluid with a dry clean cloth or tissue.
- Make sure NOT to disperse disinfection fluids into the diode. This may damage the diode.
- The removeable cover is disposable. The tips are disposable.

### 6.1 Barrier Sleeves

It is recommended to use barrier sleeves to protect the light from contamination. They generally increase the lifetime of the unit. Use the open barrier sleeve when light is used with tips, and the closed barrier sleeve when the light is used without a tip.

#### BARRIER SLEEVES OPEN

For use with cover and tips

PR00LMGU

#### BARRIER SLEEVES CLOSED

For use without cover and tips

PR00LMSL

## 7. SERVICE AND WARRANTY

The Lumina Max LAD loght is covered by 1 year warranty.

### Caution

The Lumina Max LAD light must only be serviced and repaired by the distributor Dentalica SpA.

## 8. DISPOSAL OF WASTE PRODUCTS

The tips and covers should be disposed as regular waste. The handpiece, docking station and power supply shall be disposed to a recycling central.

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La lampe Lumina Max LAD doit uniquement être entretenue et réparée par Dentalica SpA.
DENTALICA S.p.A. Via Rimini 22 | 20142 Milano - Italia